

URISAFE®

24-HR URINE COLLECTION CONTAINER



Container made of high-density polyethylene
Cap made of polypropylene

URISAFE® Urine Collection Containers are USER FRIENDLY to both patients and laboratory personnel. The flow rate is controlled for better handling of poured volumes and reduced risks of aerosol contamination. It eliminates splashing and exposure to hazardous body fluids. The low form design makes the Urisafe® very convenient for refrigerator storage by eliminating wasted space. Thanks to the large central handle, it can be gripped comfortably with three fingers while permitting pouring off of samples with ease and reduces fatigue. Both cap and container can be subjected to freezing, thawing and EtO gas sterilisation without causing changes in materials or physical appearance.

Offered in 3L and 4L sizes.



On the 4 liter model, urine can be poured into a tube by simply tilting the container forward without having to lift it.




Large centrally located handle for good balance and ease of manipulation

Cat. #	Vol. (Liters)	Dimensions (cm)	Qty/Cs
B350	3	11.5 x 24.5 x 16.0 H	40
B350-4L	4	11.5 x 24.5 x 20.1 H	30

For IVD use

LARGE OPENING FOR EASE
OF SAMPLE COLLECTION

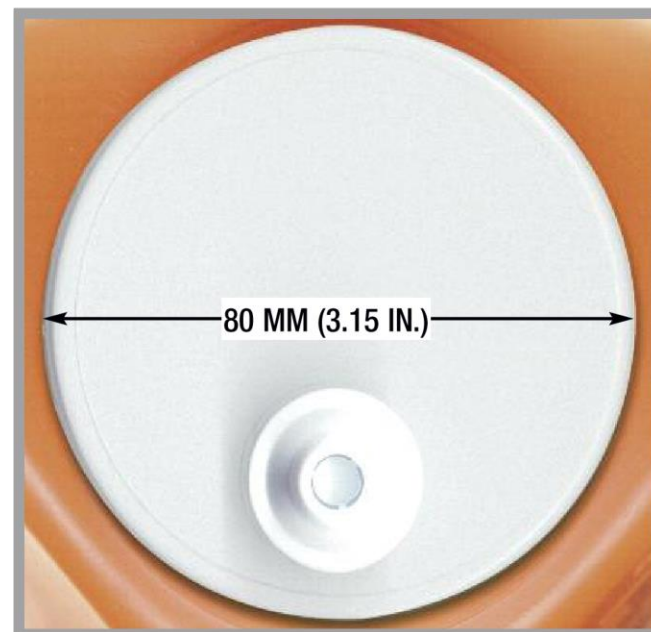

These models are also available as a biodegradable version.
Please contact Simport® for details.

URITAINER™

24-HR URINE COLLECTION CONTAINER



- 1 Large handle can be gripped comfortably
- 2 Both cap and container can be subjected to freezing, thawing and EtO gas sterilisation
- 3 Metal, latex and fluorescence free
- 4 The unique snap valve pour spout eliminates splashing and exposure to hazardous body fluids
- 5 Volume level read in 50 ml increments

Container made of high-density polyethylene
Cap made of polypropylene

The URITAINER™ offers some of the great features of the URISAFE® 24-Hr Urine Collection Container. It incorporates the same large 80 mm screw cap offering a unique snap valve pour spout and a controlled flow rate for better handling of poured volumes. It eliminates splashing and exposure to hazardous body fluids while reducing the risks of aerosol contamination when pouring. A large handle can be gripped comfortably, allowing pouring off of samples with ease and reduces fatigue. Can be subjected to freezing, thawing and gas sterilisation without modifying or changing chemical resistance or physical appearance.

Available in 2 sizes: 2.5 L and 3.5 L.



Large handle can be gripped comfortably



On all models, the unique snap valve pour spout is easily popped open and offers dripless pouring.

Cat. #	Vol. (Liters)	Dimensions (cm)	Qty/Cs
B360-25	2.5	13.3 x 13.3 x 23.2 H	40
B360-35	3.5	13.3 x 13.3 x 29.5 H	24

For IVD use

Labels for URITAINER™ & URISAFE® Containers

- Three optional labels are offered
- Patient identification label
 - Patient instruction label
 - Caution label

PATIENT AND TEST INFORMATION FOR 24-HOUR URINE COLLECTION

Patient Name: _____

Ref. #: _____

Test Required: _____

Preservative Added: _____

of Bottles Used: _____

Total Volume Collected: _____

Collection Starting Times: Hour _____ Date _____

Collection Ending Times: Hour _____ Date _____

B350-4

Patient Instructions for Collecting a 24-Hour Urine Specimen

Important: To insure accurate test results, please follow these instructions carefully.

A) Ask your laboratory whether or not you should refrigerate this bottle during the collection period.

B) At the hour you choose to start the collection period, urinate into a toilet and flush as usual.

C) Record the starting time and date in the space provided below.

D) For the next 24 hours, collect all your urine in this bottle.

E) Be sure to urinate just when the 24 hour collection period ends and include this urine in the bottle.

F) Record the ending time and date in the space provided below.

G) Promptly bring the bottle back to the laboratory.

Collection Starting Time: Hour _____ Date _____

Collection Ending Time: Hour _____ Date _____

B350-5

CAUTION!

This bottle contains a strong preservative chemical. Do not breathe vapor. Do not spill or get on skin.

B350-6

Cat. #	Description	Qty/Roll	Qty/Pk
B350-4	Patient identification label	100	1000
B350-5	Patient instruction label	100	1000
B350-6	Caution label	100	1000



B350-4HCL
Urisafe with HCl Preservative

This model offers the convenience of being pre-filled with 30 ml 6N HCl. All patient instructions are clearly detailed on label. Each container is individually wrapped.

Cat. #	Vol. (Liters)	Dimensions (cm)	Qty/Cs
B350-4HCL	4	11.5 x 24.5 x 20.1 H	30

Uritainer is a trademark of Simport® Scientific.
Urisafe is a registered trademark of Simport® Scientific.

Simport®
Scientific

2588 Bernard-Pilon
Beloeil, Qc J3G 4S5 Canada

Phone: (450) 464-1723
Fax: (450) 464-3394
Email: info@simport.com
Website: www.simport.com

Distributed by:

Printed in China 03/2015 MKT-B350LL1P4

Simport®

A Family Owned Company Since 1975



www.simport.com